

Set up MD.2 After it has been Improperly or Inadvertently Powered OFF

- Turn on MD.2 (slide switch in rear of MD.2)
- Wait to hear “Ready for Setup, System OK”
- Use PURGE to empty machine of medication cups (see PURGE MD.2 instructions, pg 9).
- Use SCHEDULE to set up your MD.2 (see MEDICATION SCHEDULING instructions, pg 4)
- Reload medication cups in machine using LOAD (see LOAD MEDICATION CUPS instructions, pg 5)

Early Access to Medications

If the user is going to be away from the MD.2 during normal medication delivery time(s), the caregiver should use the early dose feature (see EARLY DOSE instructions, pg 10)

Extended Time Away from MD.2

If the user is going to be away from the MD.2 for an extended period of time, the Caregiver should:

- Empty machine of medications (see UNLOAD MEDICATION CUPS, pg 8, or PURGE MD2 instructions, pg 9)
- Turn MD.2 OFF (USE POWER SWITCH: DO NOT JUST UNPLUG WALL TRANSFORMER)
- Contact Support Center, let operator know about the extended away time

Clear a Missed Dose

- Remove missed medication cups from storage compartment behind keypad
- Use STATUS (see STATUS instructions, pg 6) to review doses missed
- Press RUN to return to normal operations

Check How Many Days of Meds are left in MD.2?

- Use STATUS (see STATUS instructions, pg 6) to see how many days of medications remain in the MD.2

General Information


Medication Cup Capacity

10 Days @ 4 to 6 delivery times per day
20 Days @ 3 delivery times per day
30 Days @ 2 delivery times per day
40 Days @ 1 delivery time per day

Battery

Type Used: C cell, Alkaline, Quantity 8
Life: - 12 hours minimum continuous load
(18 hours typically)

Electrical Connections and Controls

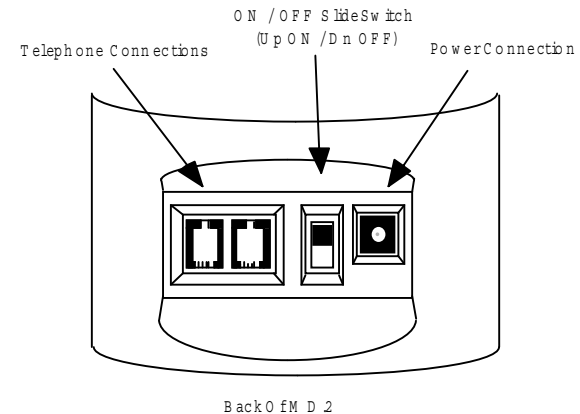

Medication Message List

- 0 - No Message -
- 1 Time for your Insulin
- 2 Take meds with food
- 3 May cause drowsiness
- 4 No food w/meds for 2 hrs
- 5 Take extra fluids w/meds
- 6 Change Catapres Patch
- 7 Change Estrogen Patch
- 8 Change Duragesic Patch
- 9 Put On Nitro Patch (*note 1*)
- 10 Remove Nitro Patch (*note 1*)
- 11 Use Nasal Spray
- 12 Use Mouth Inhaler
- 13 Take on empty stomach
- 14 No alcohol with meds
- 15 Don't drive with this med
- 16 Take meds with juice
- 17 Take meds with milk
- 18 Time for your Eyedrops
- 19 Check blood sugar level
- 20 Check blood pressure
- 21 Use your nebulizer
- 22 Remember your meal
- 23 Take your liquid meds (*note 2*)

Note 1: Message is "Change Nitro Patch" on some machines

Note 2: Message not available on all machines

How to Set up Your MD.2 for the First Time

- Connect MD.2 to a telephone line
- Plug in wall transformer and connect to MD.2
- Remove packaging from behind door
- Turn on MD.2 (slide switch in rear of MD.2)
- Wait to hear "Ready for Setup, System OK"
- Prepare medication cups
- Register MD.2 with the IMD Support Center (FAX, telephone call, or on-line @ www.imd2.com)
- Use SCHEDULE (see MEDICATION SCHEDULING instructions, pg 4) to set up machine
- Use LOAD (see LOAD MEDICATION CUPS instructions, pg 5) to load medication cups in machine

How to Load Medication Cups into MD.2


- Push LOAD (see LOAD MEDICATION CUPS instructions, pg 5)
- Follow directions on MD.2 screen exactly

How to Change Medications in Medication Cups

(Cups loaded in machine)

- Start REVIEW to gain access to each days medication (see REVIEW instructions, pg 7).
- Remove 1st days worth of medication cups (from LOAD port behind keypad). Adjust contents (ie. change meds) and put cups back in machine in EXACT ORDER they were removed
- Continue with REVIEW until all medications have been corrected

MEDICATION SCHEDULING






 Medication Scheduling
Support Center Other



Outside Line Access
Direct Dial Prefix -

 For Direct Line

OR

  Until Prefix is correct
 

Outside Line Access
Direct Dial Prefix 9



Calling Support Center
 Please Wait

Note: You will hear a dial tone, followed by dialing tones & computer tones. IF YOU DO NOT HEAR THIS, CHECK TELEPHONE CONNECTION

Unit Empty Ready To Load
 10/11 System OK 11:26A

Note : Correct Date & Time should be on Screen

PURGE MD.2

(Empty machine of meds)



Special Help Functions
Unload Purge S/N Quit

Special Help Functions
 Unload **Purge** S/N Quit



Medicine Cups in Motion
 Please wait

Unload All Cups
 Press OK Key When Done

Remove All Cups from the Load port
 (when machine stops)



Continue Unloading Meds?
Continue Quit

 To Continue

OR

Continue unloading Meds?
 Continue **quit**

 To Quit

Empty next Location

EARLY DOSE



Ready For Early Dose ?
Yes Cancel



Fetching Meds Scheduled
For 1:30 P Tue 12/18

Medication cup will be delivered

How to Correct a Machine Error; Jammed Cup (“Machine Error, Service” on screen)

Indicator:

When you see the lights flash 3 or 4 times each minute:

Cause:

A Cup is jammed

Fix:

- Free jammed cup, usually from cup delivery chute
- Power down MD.2, wait a moment & power up
- After “System OK , Ready for Setup”, use PURGE to empty machine of medication cups (see PURGE MD.2 instructions, pg 9)
- Use SCHEDULE (see MEDICATION SCHEDULING instructions, pg 4) to set up your MD.2
- Reload medication cups in machine using LOAD (see LOAD MEDICATION CUPS instructions, pg 5)

Future Cup Jam Avoidance Hints

A machine error occurred because of the following:

- Lid was not on securely:
 - Double check lids are on tightly before loading cups
- Cups were doubled up:
 - Double check cups are not stuck together before loading
- Cup was deformed beyond use:
 - Only use cups that are in good condition

STATUS (Normal Operation)



Days Until Empty 10
Battery OK

If PRN in use

Prn Doses Loaded: 09

**STATUS (Clearing/Reviewing Missed Dose(s)
or How to stop System OK from blinking)**




Remove Missed Dose Cups
Press OK Key When Done

Remove All missed dose cups



Total Doses Missed 02
Dose 1 10/11 1:30P

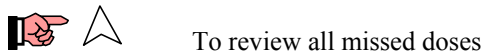

To review all missed doses



When Done

REVIEW (Medication Cups in MD.2)



Select Loading Mode
Load Review



Select Loading Mode
Load *Review*



Medicine Cups in Motion
Please Wait

Check 3 Cups Fri 07/02
Continue Quit Cup 1-3



To review next day
Meds

Next Day

OR




Check 3 Cups Fri 07/02
Continue *quit* Cup 1-3



To quit

UNLOAD MEDICATIONS CUPS

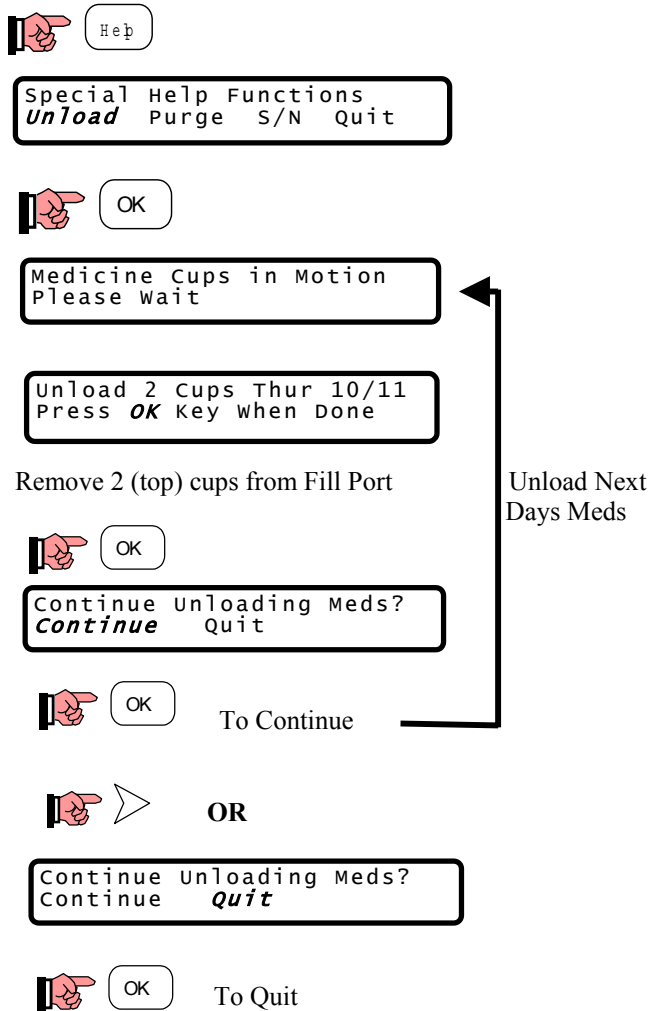

LOAD MEDICATION CUPS (For the 1st Time)

