

Special note – **SETUP FOR Med Ready MODEM SYSTEM**

1 – Go to *medready.net* on the internet and click on *setup new accounts*. Fill in the appropriate information and, allowing two business hours, follow the steps below.

The information required is:

- 1 - user's (client's) phone number
- 2 - number(s) to be call if there is a problem
- 3 - credit card or other authorization code

2 - Plug the Medready into the telephone wall jack using the cord provided. There are two telephone jacks located on the back of the unit. You may use either jack plugging the cord from

Medready into the **LINE** receptacle of the telephone.

3 - To initiate recognition of the MedReady unit to the CONTROL PROVIDER press and hold **Alarm 1**

and then press **Alarm 2** on the **MedReady** unit. Press the buttons until a light comes on below alarm buttons **3** and **4** under the **X**

4 – .The website can be accessed at: www.medready.net , then using your password and user's

(client's) # , verify the setup is complete. **If there is a problem or questions, call customer service at 310-328-7557 for additional instructions.**

Please include your 5 digit serial number before your name. for example:12345John Doe